SERION

Rf-Absorbens

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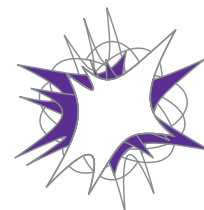
SERION

Rf-Absorbent

SERION Rf-Absorbent is recommended for the removal of IgM rheumatoid factors from human serum and plasma samples prior to pathogen-specific IgM detection.

Rheumatoid factors are predominantly anti-IgG directed autoantibodies of the IgM class that can be detected in serum samples of approximately 5 % of healthy individuals. Those IgM rheumatoid factors preferably bind to IgG immune complexes. Therefore, false positive results may appear in pathogen-specific IgM detection. In samples with a high concentration of pathogen-specific IgG antibodies, weakly binding pathogen-specific IgM antibodies may be displaced by stronger binding IgG antibodies. In those cases, IgM detection can show false negative results. Therefore, it is necessary to pretreat serum samples with rheumatoid factor-absorbents. Prior to pathogen-specific IgM detection, it precipitates up to 15 mg IgG / ml undiluted serum and thereby also IgG binding IgM rheumatoid factors.

Order Information

Rf-Absorbent (20 ml for 100 determinations)

Order Nr.: Z200

Please visit our website www.virion-serion.com for more information on our products.